

# SHOWER KIT 1

Tiles: 16 pcs.

Width: 25 x 25 cm Height: 18 mm

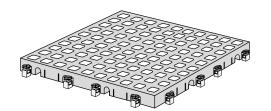

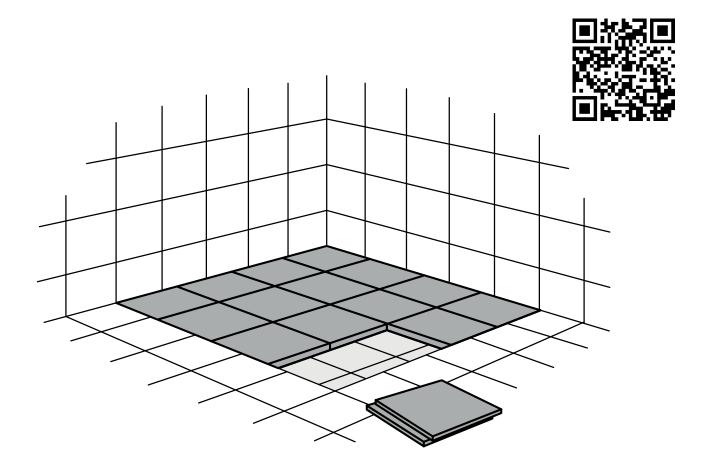




USE ONLY FOR SHOWERS UP TO 100X100 CM. FOR BIGGER SHOWERS, USE TWO KIT 1'S.

ALWAYS FILL UP THE WHOLE SHOWER AREA TO AVOID ACCIDENTS.






## **IMPORTANT GUIDELINES**

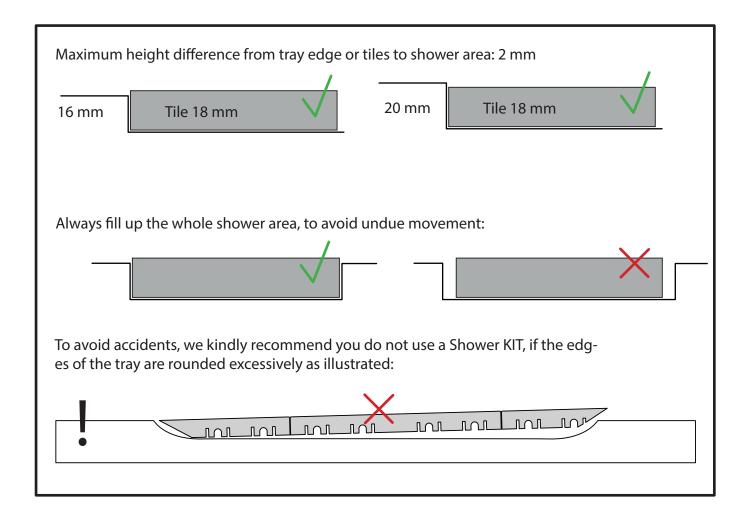


## **SHOWER KIT 1 CONTENT**

Tiles: 16 pcs. Width: 25 x 25 cm Height: 18 mm

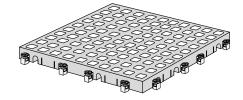


Shower hook + shower support




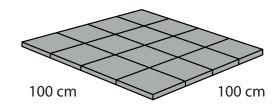


## **SHOWER KIT 1 INSTALLATION**

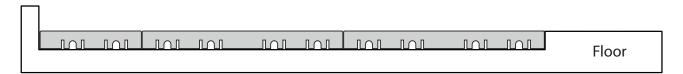


Use Shower KIT 1 if the recess area fits the height of one tile.

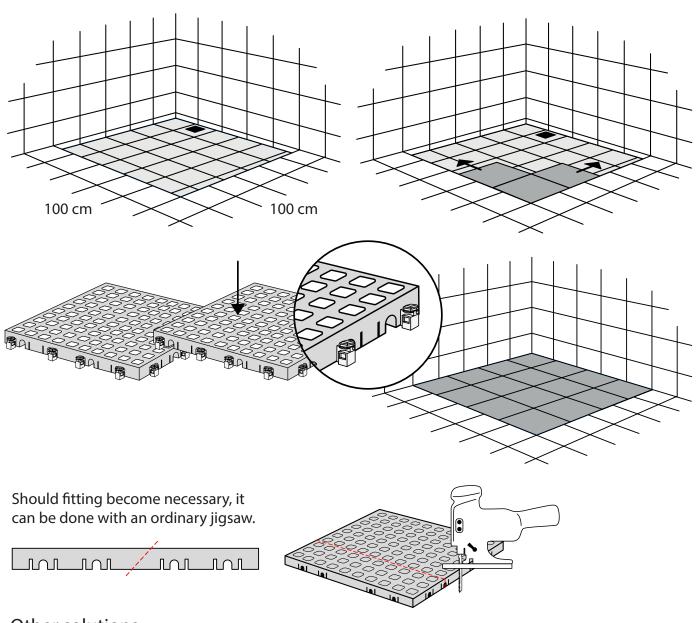


#### Other solutions

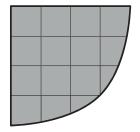

Rounded shower.

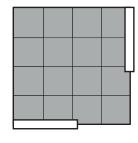

Shower with walls and sloping edges.

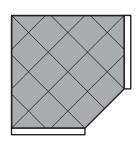



Shower area with walls and a sloping edge.

#### **CLEANING OF SHOWER KIT**


The surface of the tiles and ramps have no pores. And thus cannot absorb bacteria. Water and soap can easily exit, through the holes in the tiles.

We recommend cleaning the tiles every four weeks in ordinary use. The tiles tolerate all types of conventional cleaning agents. They also tolerate autoclaving (+130° C) to be sterilized.

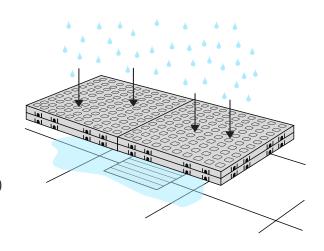

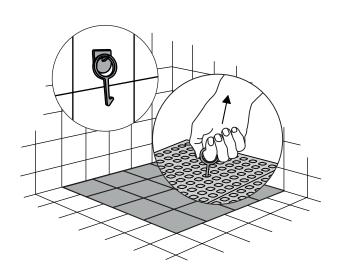

"Shower hook" makes it easy to lift the tiles in the shower.

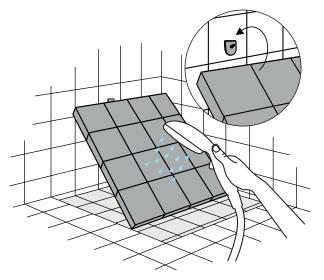

"Cleaning support" mounted on the wall, support the tiles while cleaning.

#### **DISPOSAL**



KIT and KIT parts: These products are made from PE (Polyethylen). For disposal after use we encourage you to make sure that the KIT is either returned to the point of purchase for collection and shipment back to Excellent (for regeneration), or sorted accordingly to the recommendations in your local area.



Plastic bag: Recycled as clear PELD plastic.



Cardboard box: Recycled as cardboard.



Brochure and instruction: Recycled as paper after end use.



sales@ex-as.com | Møllevej 2 - 8544 Mørke - DK | +45 86377133